0123

ConvaMax™ SUPERABSORBER

Superabsorber adhesive dressing / Superabsorberende - klæbende bandage / Högabsorberande vidhäftande förband / Superabsorber selvklebende bandasje / Superimevä kiinnittyvä sidos / Superupijajuča ljepljiva obloga / Superabsorbėjošs pašlipošs pārsejs / Itin gerai absorbuojantis lipnus tvarstis / Superabsorbeeruv kleepuv haavaside / Superabsorpcní adhezivní krytí / 超强吸收力黏性敷料 / 超強吸收力黏性敷料

INSTRUCTIONS FOR USE / BRUGSANVISNING / BRUKSANVISNING / BRUKSANVISNING / KÄYTTÖOHJEET / UPUTE ZA UPORABU / LIETOSĖJIMAS PAMACIBA / NAUDOJIMO INSTRUKCIJA / KASUTUSJUHEND / NÁVOD K POUŽITÍ / 使用說明 / تعليمات الاستخدام

ENGLISH

Description: ConvaMax™ Superabsorber adhesive dressing with a silicone layer is a sterile dressing for use on highly exuding wounds, consisting of a breathable and waterproof nonwoven layer, a Superabsorber pad layer, a fabric wicking layer and soft silicone wound contact layer. The multi-layered construction facilitates fluid management to provide the optimal moist wound environment which leads to the promotion of faster wound closure. The absorbent pad provides superior absorption and locks exudates away. This minimizes the risk of maceration and damage to peri-wound skin.

Indications: ConvaMax™ Superabsorber adhesive dressing is indicated for management of moderate to highly exuding wounds, including: leg ulcers, pressure ulcers, diabetic foot ulcers and dehisced surgical wounds.

Directions for use:

- After the wound is cleansed, the appropriate size dressing is selected to allow for sufficient space to the wound edge.
- ConvaMax™ Superabsorber adhesive dressing can be used both as a primary dressing or as a secondary dressing.

When used as a primary dressing, ConvaMax™ Superabsorber adhesive dressing must be in close contact with the wound bed to ensure wound fluid is absorbed. White contact layer should be in contact with the skin and blue waterproof backing away from the body.

- ConvaMax™ Superabsorber adhesive dressing is self-adhesive but may require additional support with a fixation dressing or mild-to-moderate compression bandaging.
- The frequency of changing dressing depends on the level of exudate.
- Dressing removal can be facilitated by moistening the dressing with saline solution.
- Dispose of as clinical waste. If contaminated, follow local regulations.

Contraindications:

- Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption. Do not use on third-degree burns.
- ConvaMax™ Superabsorber adhesive dressing should not be used on patients with a known allergy or hypersensitivity to the dressing or its components.
- Arterial bleeds and heavily bleeding wounds.

Caution:

- Do not reuse. Do not use the dressing if the sterilized packaging is damaged or open.
- Do not cut the dressing, the cover must remain intact.

Components:

Polypropylene, Sodium polyacrylate powder absorbent pad, Polyurethane coated with Silicone gel & Polyethylene film.

Storage: Please store the product in cool and dry place and avoid light

Shelf life: 3 years from date of manufacture

© 2022 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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Distributed by:
 ConvaTec Limited
 First Avenue, Deeside Industrial Park
 Deeside, Flintshire, CH5 2NU, UK
 ConvaTec Inc.
 Greensboro, NC 27409
 USA

■ Huizhou Foryou Medical Devices Co., Ltd
 North Shangxia Rd, Dongjiang Hi-tech Industry
 Park, 516005, Huizhou, P.R. China
 Tel: +85-752-5302186 Fax: +85-752-5302020

■ Shanghai International Holding Corp.
 GmbH (Europe) Effertstrasse 80
 20537 Hamburg, Germany

ConvaTec Inc. has granted a license to Huizhou Foryou Medical Devices Co., Ltd. to use the ConvaMax and ConvaTec trademarks.

ConvaTec Middle East FZ-LLC
 Dubai Science Park,
 HQ Complex, North Tower – 604N
 Dubai, UAE
 Tel: +971501075566

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Subsidiary Office
 ConvaTec Australia Pty Ltd
 Subsidiary Office
 Level 1, 5 Nexus Court
 Nexus Corporate park
 Mulgrave, VIC 3170

Subsidiary Office
 ConvaTec Finland Oy
 Keilaranta 168
 4th Floor (B4.r)
 FI-0267 Espoo
 Tel: +358 20 7659 630

Subsidiary Office
 ConvaTec Norge A/S
 Nils Hansensvei 8
 N-0667 Oslo
 Tel: +47 21 09 67 90

Subsidiary Office
 ConvaTec Czech Republic
 Svatoplukova 10
 150 00 Praha 5
 Tel: +420 224 12 12 12

Subsidiary Office
 ConvaTec Korea Ltd.
 1000-100, 10th Floor
 1000-100, 10th Floor
 1000-100, 10th Floor

Subsidiary Office
 ConvaTec Mexico S. de CV
 Av. de las Industrias 1000
 04500, Toluca, Mexico

Subsidiary Office
 ConvaTec Spain S.L.
 Calle de la Industria 1000
 04500, Toluca, Mexico

Subsidiary Office
 ConvaTec Taiwan Ltd.
 1000-100, 10th Floor
 1000-100, 10th Floor
 1000-100, 10th Floor

Subsidiary Office
 ConvaTec Thailand Ltd.
 1000-100, 10th Floor
 1000-100, 10th Floor
 1000-100, 10th Floor

SI2433A

Issued 2022-06