Gelierungssachets und Sachets für Geruchskontrolle / Sachets de gélification et de contrôle des odeurs / Bustine gelificanti anti-odeur / Sobres de gelificación y control del mal olor / Geleer- en geurbeheersingssachets / Gelatinerade och luktkontrollerade doftpåsar / Gelérés- és luktgéltáplálék / Gel- og lugtkontrollposer / Hydyttys- ja hajunhallintapussit / Saquetas de control de odor e gelificante / Sáčky na gelování a potlačení zápachu / Φακέλακια ζελατινοποίησης και ελέγχου οσμών / ialis おとしのい 抑制小袋 / اکساص صغيره للتنفس والتحكم في الروائح / Jellemezse ve Koku Kontrol Poşetleri / Çawie c temepem de kontrolla zapaxha / Sachets de gélification et de contrôle des odeurs / Saszetki żelujące i redukujące nieprzyjemny zapach / Vrećice za geliranje i kontrolu neugodnih mirisa / Geelistavat löhnaemaludskotikesed / Zselésítő szagtaláló tasakok / Želejošas un smakas kontrolejošas pacīnas / Suširistimo ir kvapo kontrolės paketeliai / Կեսիկա գոլորան և կոնտրոլ համար / Plicuri pentru gelifiere și reducere miroslului / Gélujúca de dezodorácie vrecúška / Zavojčki za geliranje in uravnavanje vonja

en ENGLISH

Product Description: Gelling and Odour Control Sachets

Intended Use: Convatec Diamonds™ are intended to gel the contents of an ileostomy or Colostomy pouch, or Fecal management system collection bag, and to help reduce or eliminate unwanted odours and excess flatus.

Indications for use: For use with semi-formed to liquid stool

Intended users: Health Care professionals, Caregivers and Ostomates

Instructions for use:

- 1- Place one complete sachet into your pouch. Do not open sachet.
- 2- Empty and dispose of the pouch contents as normal. Additional sachets can be added through the pouch outlet after emptying.

Please note: Your output may appear darker than usual and/or may contain black flecks. This is a visible indication of the ActiveOne™ Odour Control system. If monitoring output colour, temporarily discontinue use.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

Warnings and Precautions: Keep out of reach of children. Do not eat. Use only as directed. For external use only. In case of contact with eyes, rinse immediately with clean water and seek medical advice. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations. If during the use of this device or as a result of its use a serious incident has occurred please report it to the manufacturer and to your national authority.

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