
Eurotec

Instructions for use 2-piece ostomy system Combimate™ / Gebrauchsanleitung für die zweiteilige Stomaversorgung Combimate™ / Istruzioni per l'uso del sistema per stomia a 2 pezzi Combimate™ / Notice d'utilisation du système 2 pièces de stomathérapie Combimate™ / Instrucciones de uso del sistema de ostomía de 2 piezas Combimate™ / Instruções de utilização do sistema de ostomia de 2 peças Combimate™ / Gebruiksaanwijzing 2-delig stomasysteem Combimate™ / Bruksanvisning 2-dels stombandagge Combimate™ / 2-osainen Combimate™ -avanesidosken käytöohjeet / Brugsanvisning til Combimate™ 2-delt stomisystem / Bruksanvisning 2-dels stomsystem Combimate™ / Οδηγίες χρήσης οστούριου σπινός 2 τεμαχίου Combimate™ / Használati utasítás - Combimate™ 2-részes sztomatérápiás segédeszköz rendszer használatához / Instrukcja używania 2-częściowego systemu stomijnego Combimate™ / Návod k použití dvoúdielneho stomického systému Combimate™ / Návod na použitie 2-dielneho stomického systému Combimate™ / Navodila za uporabo dvočelnega sistema za stomio Combimate™ / Инструкци по применению двухкомпонентной системы для стомы Combimate™ / Instrucțiuni de utilizare pentru sistemul stomic din 2 componente Combimate™ / 2-osallise stomisistemin Combimate™ kasutusjuhend / Lietošanas pamācība divdaļīgajai stomas sistēmai Combimate™ / Navodilno instrukcijs dviejų dalių stomos sistemai „Combimate™“ / Инструкција за употребу на 2-компонентна система Combimate™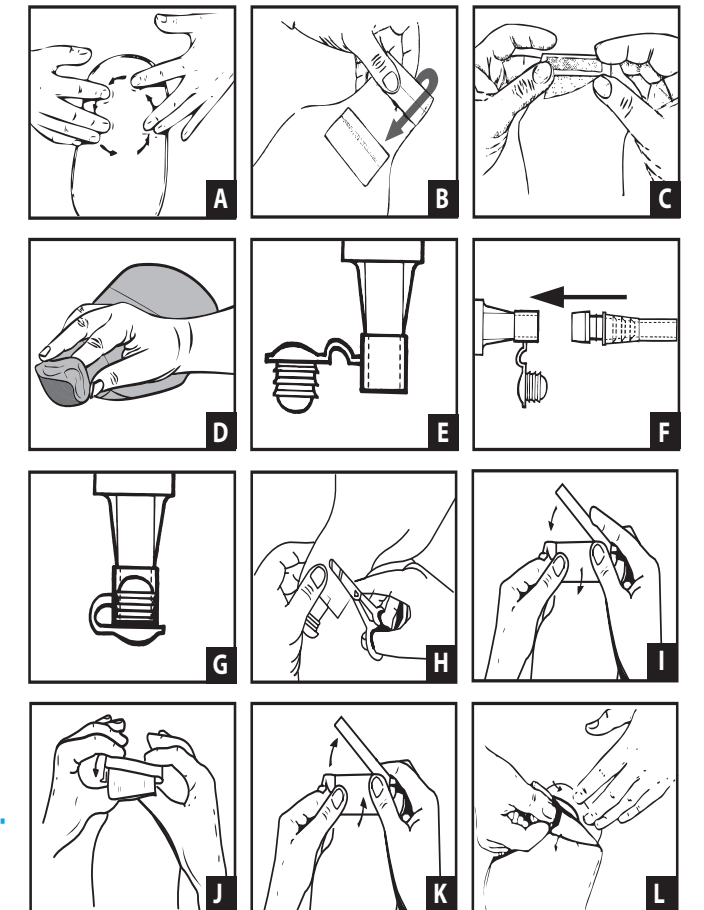

ENGLISH

Intended Use
Management of stomaal output.
Indications for use
For use with a colostomy, ileostomy or urostomy. To be used in combination with Combimate™ 2-piece system wafers.
Contra indication
Do not use on individuals who are sensitive to or who have developed sensitivity or allergic reaction to the product.
Precautions and Observations
Should irritation occur, discontinue to use and consult your healthcare professional.
Instructions

1. Applying the pouch
Separate pouch flims to allow air to enter. Line up the skin barrier and pouch flange, and press on the bottom of the flange. Then move to the top of the flange, pressing as you go around (A). Gently pull-on the pouch to ensure secure connection. Wear as long as comfortable and secure. Wear time may vary.
2. Closing, opening, and emptying of the pouch
A. Drainable Pouches
A.1 Pouches with Velcro closure
The drainable pouches have a SOFTSAFE™ fastening with a 'finger pocket'. The SOFTSAFE™ fastening can be closed easily by folding the end several times (B and C). Then, briefly press it all down.
The SOFTSAFE™ fastening can be opened by unfolding the Velcro and rolling it open. To open the bottom of the pouch, put one finger in the sleeve (the 'finger pocket') (D) and two fingers to the sides of the pouch. Use the other hand to clean approximately 1 cm of the inside of the pouch.

A.2 Drainage tap (high output pouches)
High output pouches have a drainage tap with a soft plug (E), that can be connected to a drainage bag by means of an adapter (F). The plug needs to be fully inserted into the tap (G). If desired, you can cut off the drain tap right above the tap (H) and close the pouch with the clip provided.
Fold the tail of the pouch only once over the narrow part of the clip (I). Avoid creating any folds. The clip can be closed by snapping the two sections into each other (J).
The clip can be opened by unsnapping the two sections (K)

B. Urostomy Pouches
The urostomy pouches have a bottom tap with a soft cap (E). The cap must be pushed fully into the drain opening (G). An additional strip is attached to the front, in which you can fold the bottom tap, stopping it from touching the skin directly.
Connection to Night Drainage Tap:
The separately supplied adapter (F) can be used to connect the pouch to a night drainage bag. Under no circumstance should you insert the conical tip of the night drainage bag directly to the bottom tap.
3. Information about Pouches with Filters (applicable for colostomy pouches only)
It filter is present on the pouch, cover it with the sticker provided when bathing, swimming, etc. Remove after water activity.
4. Removing of the pouch
Support the skin wafer and carefully pull the tab on the top side of the flange forward (L). Clean the stoma and the area inside of the flange with lukewarm water before applying a new pouch

5. Disposal
After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.
This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.
If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.
© 2024 Convatec Inc.
**/ All trademarks are the property of their respective owners.

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© 2024 Convalec Inc.