STOMADRESS® PLUS

One-Piece Drainable Pouch

Instructions for Use

INTENDED USE

Management of stomal output.

INDICATIONS FOR USE

For use with a colostomy or ileostomy.

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

POUCH PREPARATION


The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

For pouches with a precut stoma opening:


Select a pouch with the precut stoma opening at least 1/8 in. (3mm) **larger** than the largest stoma diameter. **DO NOT enlarge the precut stoma opening.** Remove protective plastic shield from Stomahesive® disc.

For custom cut pouches:

Before cutting, separate front and back panels. Using guidelines on release paper, cut the center hole the same size as the stoma. **Do not cut beyond last line.**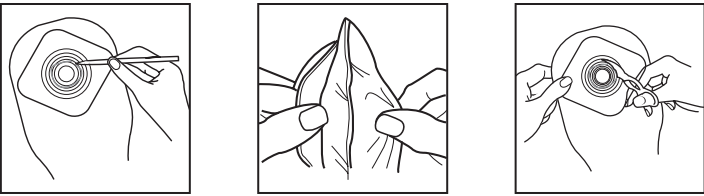


POUCH APPLICATION

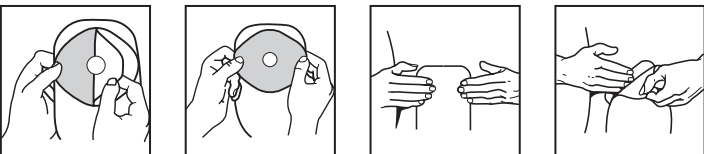Remove release papers. Ensure a snug fit around the stoma.

USING THE TAIL CLOSURE

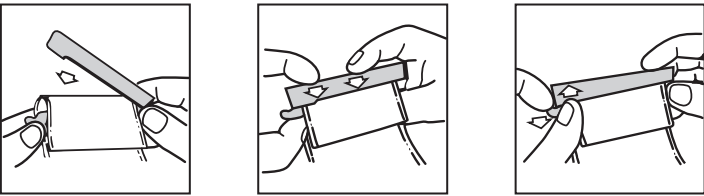POUCH REMOVAL

This product may remain in place while bathing, showering, or swimming. Pat dry after exposure to water.

DISPOSAL

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

Note: Should irritation occur, discontinue use and consult a healthcare professional.

© 2023 ConvaTec Inc. ®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Limited
EC **REP** **UNOMEDICAL A/S**
DKSH Taiwan Ltd.