SurePress®
High Compression Bandage

III Convatec

PRODUCT DESCRIPTION

The SurePress® High Compression Bandage is designed for patients with impaired venous return. If the patient has an ulcer, the SurePress High Compression Bandage should be used over a primary dressing. It is an elasticized bandage consisting of cotton/viscose, nylon and Lycra® with a yellow center line and two rectangular extension indicators for different size limbs. (See diagram under Number 4 in Instructions for Use.)

INDICATION

Venous leg ulcer management and associated conditions where compression therapy is indicated.

CONTRAINDICATIONS

Arterial and mixed venous/arterial ulcers.

Legs with an ankle circumference less than 18 cm.

As with all products, SurePress High Compression Bandage should not be used on individuals with a known sensitivity to the product or its components.

PRECAUTIONS

Initial use of this product should be under the direction of a healthcare professional.

If redness or irritation result or if toes become discolored black or blue, remove bandage and consult a healthcare professional.

Use of the SurePress High Compression Bandage directly in contact with an open wound is not recommended.

INSTRUCTIONS FOR USE

Cleansing the Wound Site

Cleanse the ulcer according to normal practice, rinse well and dry the surrounding skin prior to application of the primary dressing and the compression bandage.

Following the application of a primary dressing, a SurePress™ Absorbent Padding (supplied separately) may be used as an underlayer before the compression bandage. This soft absorbent wrap helps to distribute graduated compression and protect bony, vulnerable areas and skin from any exudate leaking from the primary dressing.

BANDAGE APPLICATION

1. Determine whether the limb size is "normal" or "large" using a tape measure. Ankle sizes between 18-26 cm are considered "normal". Ankles in excess of 26 cm are considered "large".
2. Having determined the limb size, select the small or large rectangular extension indicator on the bandage. The small rectangle should be selected for "normal" limb sizes and the large rectangle for "large" limb sizes.
3. Start the bandage at the centre of the ball of the foot with the lower edge of the bandage at the base of the toes. Wrap the bandage using minimal tension

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