CE 0123

ConvaMax™ SUPERABSORBER

Superabsorber non-adhesive dressing / Superabsorber nicht adhäsiv / Pansement superabsorbant non-adhésif / Medicazione Super-assorbente non-adesiva / Apósito súperabsorbente no adhesivo / Superabsorbčné neadhezívne krytie / Υπεραπορροφητικό μη συγκολλητικό επίθεμα / Superabsorberend niet-klevend verband / Pensó/Curativo não adesivo Superabsorvente / Opatronek z superabsorbentem, nieprzyklepny
INSTRUCTIONS FOR USE / GEBRAUCHSANLEITUNG / MODE D'EMPLOI / ISTRUZIONI D'USO / INSTRUCCIONES DE USO / NÁVOD NA POUŽITIE / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ / GEBRUIKSAANWIJZING / INSTRUÇÕES DE UTILIZAÇÃO / INSTRUKCJA UŻYCIA

en ENGLISH

Description:

ConvaMax™ Superabsorber non-adhesive dressing is a sterile dressing for use on highly exuding wounds, consisting of an air permeable and waterproof nonwoven layer with a Superabsorber pad layer. The triple-layered construction facilitates fluid management to provide the optimal moist wound environment which leads to the promotion of faster wound closure. The absorbent pad provides superior absorption and locks exudate away. This minimizes the risk of maceration and damage to peri-wound skin.

Indications:

ConvaMax™ Superabsorber non-adhesive dressing is indicated for management of moderate to highly exuding wounds, including: leg ulcers, pressure ulcers, diabetic foot ulcers and dehiscd surgical wounds.

Directions:

1. After the wound is cleansed, the appropriate size

dressing is selected to allow for sufficient space to the wound edge.

- ConvaMax™ Superabsorber non-adhesive dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, ConvaMax™ Superabsorber non-adhesive dressing must be in close contact with the wound bed to ensure wound fluid is absorbed. White contact layer should be in contact with the skin and blue waterproof backing away from the body.
- Secure accordingly with retention bandage, tapes or compression bandage if assessed as clinically appropriate.
- The frequency of changing dressing depends on the level of exudate.
- Dressing removal can be facilitated by moistening the dressing with saline solution.
- Dispose of as clinical waste. If contaminated, follow local regulations.

Contraindications:

- Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption. Do not use on third-degree burns.
- ConvaMax™ Superabsorber non-adhesive dressing should not be used on patients with a known allergy or hypersensitivity to the dressing or its components.
- Arterial bleeds and heavily bleeding wounds.

Caution:

- Do not reuse. Do not use the dressing if the sterilized packaging is damaged or open.
- Do not cut the dressing, the cover must remain intact.

Storage: Please store the product in cool and dry place and avoid light.

Shelf life: 3 years from date of manufacture

© 2021 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Inc. has granted a license to Huizhou Foryou Medical Devices Co., Ltd. to use the ConvaMax and ConvaTec trademarks.
Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80 20537 Hamburg, Germany
ConvaTec Limited First Avenue, Deeside Industrial Park Deeside, Flintshire, CH5 2NU, UK
ConvaTec Inc. Greensboro, NC 27409 USA
Importer: ConvaTec International Services GmbH Mühlentalstrasse 36/38 CH-8200 Schaffhausen