SurePress

Absorbent Padding/Bandage absorbant/Venda

Almohadilla Absorbente/Bendaggio

Assorbente/Compressa Assorbente

6 bandages/vendas/
bendaggios/bandagens
10 cm x 3 m

US - ORDER NO. 650948
CANADA - ORDER NO. 650948
BRASIL - CÓDIGO: 650948/793
UK - ORDER NO. S126
ESPAÑA CÓDIGO: 650948
ITALIA - CODICE: 650948

ConvaTec Inc.   UNOMEDICAL A/S
Greensboro, NC 27409 USA Aaholmvej 1-3, Osted
4320 LEJRE DENMARK
1-800-422-8811

 ConvaTec Limited, First Avenue
Deeside Industrial Park, Deeside, Flintshire, CH5 2NU, UK

Imported by:
DKSH Taiwan Ltd.
10th Floor, No. 22, Lane 407,
Tiding Blvd., Sec. 2,
Neihu Technology Park, Taipei City 114,
Taiwan, R.O.C

Sponsored in Australia by:
ConvaTec (Australia) Pty Ltd.
Unipark Monash, Building 2, Ground Floor
195 Wellington Road
Clayton, Victoria 3168
(P.O. Box 63 Mulgrave, Victoria, Australia 3170)

ConvaTec (NZ) LTD
Level 4
369 Queens Street
P.O. Box 62663 Kalmia Street
Auckland, New Zealand

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PRODUCT DESCRIPTION

SurePress Absorbent Padding is a soft, nonwoven, absorbent padding composed of a blend of viscose/bicomponent and superabsorbent fibers (88:12). The SurePress Absorbent Padding protects the skin, provides additional absorption and even distribution of pressure under compression bandages.

INDICATIONS

The SurePress Absorbent Padding is indicated for use under compression bandages in the management of venous leg ulcers and associated conditions.

CONTRAINDICATIONS

Compression of any kind should not be used for the management of arterial and mixed venous/arterial ulcers. As with all products, the SurePress Absorbent Padding should not be used on individuals with a known sensitivity to the product or its components.

PRECAUTIONS AND OBSERVATIONS

Use of this product should be under the direction of a health care professional. Use of the SurePress Absorbent Padding directly in contact with an open wound is not recommended.

DIRECTIONS FOR USE

Apply the SurePress Absorbent Padding over the primary wound dressing, starting at the

base of the toes, proceeding up the leg in a spiral fashion, overlapping the bandage on each turn by 50% of its width and finishing just below the knee. Ensure that bony prominences are adequately padded.

BANDAGE REMOVAL

After removal of the compression bandage, carefully unwind the SurePress Absorbent Padding and dispose. For single use.

STORAGE PRECAUTIONS

Protect from light; store in a cool, dry place.

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© 2021 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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