STERILER

Loop Ostomy Rod

Inspect devices prior to use for damage during shipment or storage or any out-of-box defects that might increase the likelihood of fragmentation during procedure.

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

Instructions for Use

For use in loop ostomy surgery.

For use with ConvaTec skin barrier and pouch components as follows (see manufacturer's instructions for use of the skin barrier and pouches):

Rod Size	Component Flange Size
65 mm 90 mm	70 mm or 100 mm 100 mm

The loop ostomy rod is designed with one swivel end.

In Position **A** the rod may be pushed through the loop of mesentery.

Once in position, turn the swivel end to Position **B**. The perforations in the triangular fittings at the ends of the loop ostomy rod permit suturing of the rod to the abdomen if desired.

CAUTION: Use extreme care during handling and operation of swivel to ensure that it is not damaged or separated from the rod.

REMOVING THE LOOP OSTOMY ROD

Return the swivel end to position **A**. Remove the rod by gently sliding the rod along the skin surface until the rod is completely free of the bowel.

CAUTION: Inspect devices immediately upon removal from the patient for any signs of breakage or fragmentation.

DISPOSAL

Adhere to local disposal requirements/regulations.

For more information on ConvaTec products, contact us at the address below.

© 2024 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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