Little Ones[®]

One-Piece Drainable Pouch with *InvisiClose*[®] Outlet

Instructions for Use

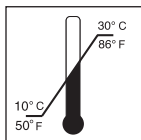
Single Use

Read instructions for use.

STORAGE CONDITIONS

Storage temperature.

Keep dry. Avoid high humidity.

Lot/control number

PRODUCT DESCRIPTION

Little Ones[®] One-Piece Drainable Pouch with *InvisiClose*[®] Outlet is a one-piece ostomy system consisting of a drainable pouch with an InvisiClose tail closure system. The Invisiclose tail closure system is comprised of a security flap, interlocking closures and outlet end-strips.

Note: Extra Small Little Ones[®] pouches do not have a security flap and should only be folded three times.

INDICATIONS

For the management of stomal output.

PRECAUTIONS AND OBSERVATIONS

CAUTION: Care should be exercised when using adhesive products around a flush urostomy, especially when associated with megaureters and the absence of a urinary reservoir. Rarely, clinical reports have been documented of the presence of adhesive in the renal collecting system of these patients. Should a mass be detected in the urinary tract, evaluate the continued use of the adhesive product and initiate appropriate clinical management.

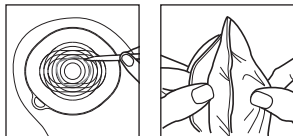
Note: Should irritation occur, consult a healthcare professional.

Care should be taken to exert only gentle pressure on the abdomen of infants when applying any pouching system.


DIRECTIONS FOR USE

The peristomal skin **must** be clean, dry, and free from any solvent or greasy substances.

FOR CUSTOM CUT POUCH PREPARATION

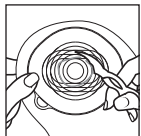Before applying the pouch, allow a small amount of air to enter it by gently spreading the pouch as shown. Leaving the release paper in place, use the guide on the release paper to trace the outline of the stoma.

Do not cut beyond last line as indicated on the release paper.

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
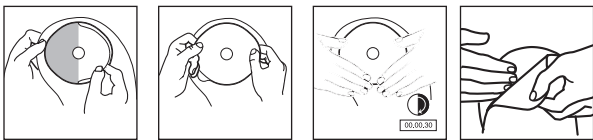
APPLICATION


Center the cut opening over the stoma and press the adhesive skin barrier into place. Carefully smooth the inner edge closest to the stoma with a finger, holding for 30 seconds, to ensure a snug fit around the stoma.

DIRECTIONS TO CLOSE THE POUCH:

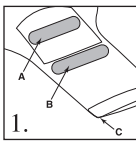
Note: Extra Small Little Ones® pouches do not have a security flap and should only be folded three times.

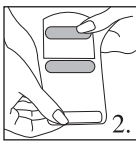
The InvisiClose tail closure system (1) is comprised of a security flap (A), inter-locking closures (B) and outlet end-strips (C).

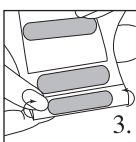
Hold the security flap against the front of the pouch revealing the inter-locking closures. (2)

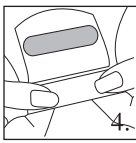Fold the tail end up four times and ensure the inter-locking closures overlap. (3)

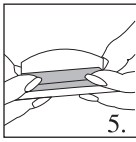
Press the inter-locking closures together until you feel them lock (CLICK) into place along their entire length. (4)

Wrap the security flap around to the back of the pouch (body side) and press the inter-locking closures together until you feel them lock (CLICK) into place. (5)

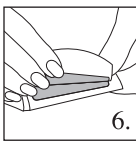DIRECTIONS FOR DRAINING THE POUCH:

Lift the pouch tail so it is facing upwards. Carefully lift the security flap up to unlock. (6)

While holding the security flap up against the front of the pouch, unlock the inter-locking closures by lifting at the corner (7) and unraveling the folds. (8)

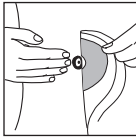
To facilitate draining and/or cleaning, open the tail end by pushing in both ends of the outlet end-strips with the thumb and forefinger. (9)

If desired, pouch tail may be cleaned with gentle soap and water. Pat dry.

REMOVAL

Drain the pouch before removal.

To remove the adhesive barrier from the skin, support the skin with one hand and gently peel the adhesive barrier away from the skin.

DISPOSAL

Adhere to local disposal requirements/regulations.

For more information on ConvaTec products, contact us at the address below.

Manufactured by:

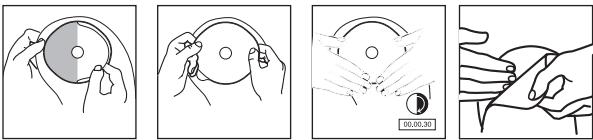
ConvaTec Inc.
Greensboro, NC 27409
1-800-422-8811

www.convatec.com

© 2024 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.

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Fabriqué par: ConvaTec Inc.
MD/MC indiquent respectivement des marques déposées et des marques de commerce de ConvaTec Inc.