
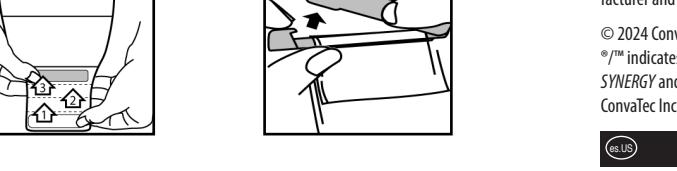
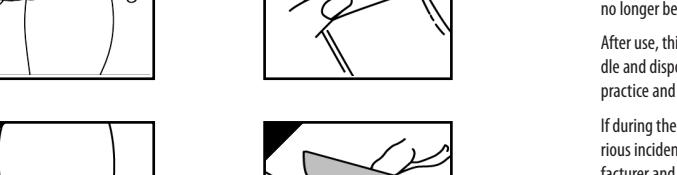
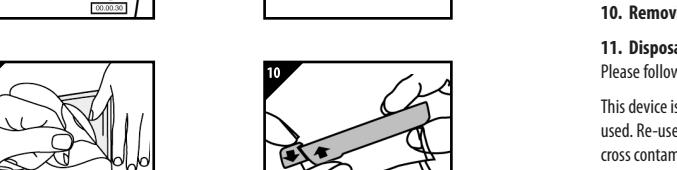
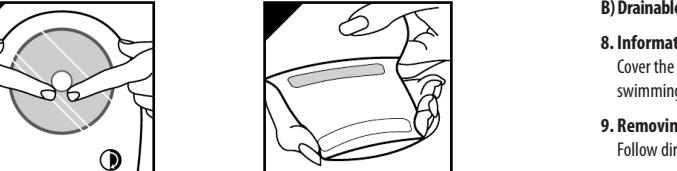
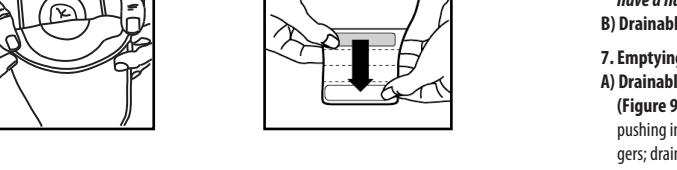
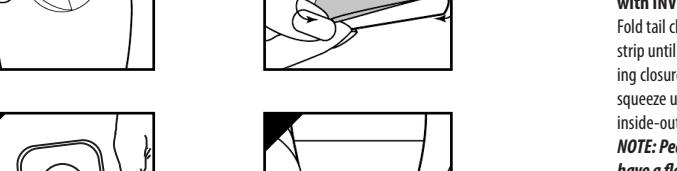
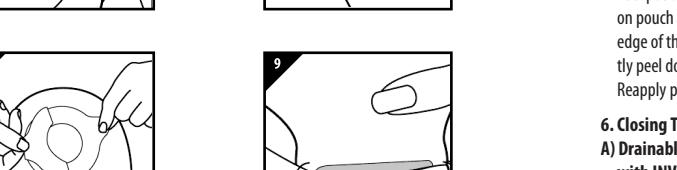
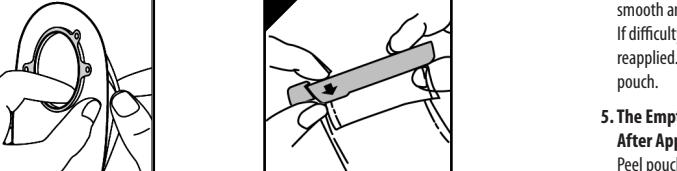
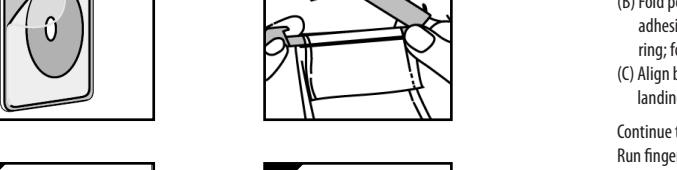
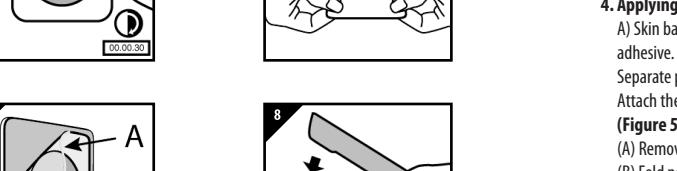
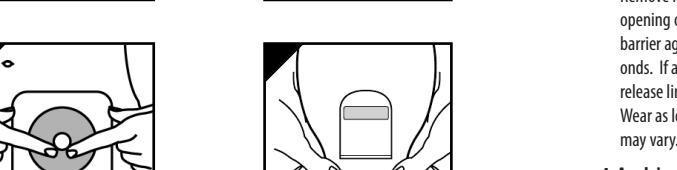
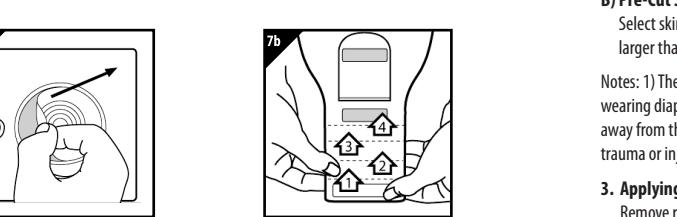
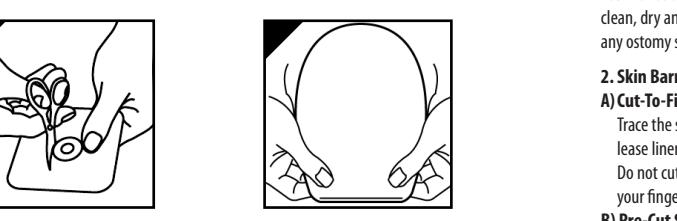
ESTEEMTM SYNERGY[™]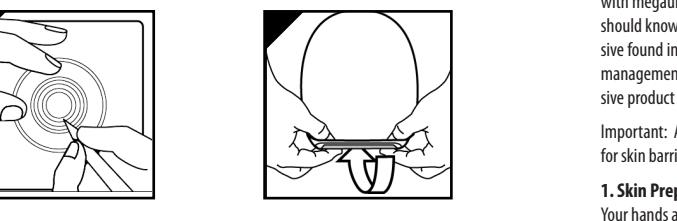

ENGLISH

Two-Piece Cut-to-Fit or Pre-Cut Ostomy System with ADHESIVE COUPLING TECHNOLOGY™

INSTRUCTIONS FOR USE
(Must be used with the same size ESTEEM SYNERGY™ pouch)

Intended Use: Management of stomal output.

Indications for Use: For use with a colostomy or ileostomy.

PRECAUTIONS AND OBSERVATIONS:

Caution: Patients with a flush ostomy, when associated with megaoesophagus and the absence of a urinary reservoir, should ensure that there have been reported cases of adhesive failure or dehiscence before applying ostomy products.

Important: A medical professional should assess the need for a skin barrier prior to use.

1. Skin Preparation

Your hands and the skin surrounding the stoma must be clean, dry and free from oily substances before applying an ostomy system.

2. Skin Barrier Preparation

A) Cut-to-Fit Skin Barrier (Figure 1)
Trace the shape of the stoma onto the skin barrier in a media line. Seal the skin barrier just outside the tracting area. Do not cut beyond last line. Smooth cutting open with your fingers.

B) Pre-cut Skin Barrier

Select skin barrier with pre-cut stoma opening slightly larger than your stoma.

Notes: (1) The stoma collar may cause skin irritation if worn during diapering. If irritation occurs, remove the collar away from the skin barrier. (2) Protect the stoma from trauma or injury.

3. Applying the Skin Barrier (Figure 2)

Remove release film from skin barrier. Center the opening of the skin barrier onto the stoma for 30 seconds. At the end of the 30 seconds, gently press the adhesive ring onto the skin barrier. Retain the etiquette adhesive during reapplication.

4. Insertion of the skin barrier (Figure 3)

With adhesive ring facing outwards, place the skin barrier onto the skin around the stoma for 30 seconds. At the end of the 30 seconds, gently press the adhesive ring onto the skin barrier. Retain the etiquette adhesive during reapplication.

5. Removal of the skin barrier (Figure 4)

With adhesive ring facing outwards, gently pull the skin barrier away from the stoma. Remove the adhesive ring.

6. Cleaning the pouch (Figure 5)

Fold pouch closure beginning with the white outer edge strip until interlocking closure opens. Press interlocking closure together across the width of the stoma. Open the pouch.

7. Preparing the pouch (Figure 6)

Separate pouch film to allow air to enter. (Figure 4) Attach the matching pouch to skin barrier as follows:

(Figure 5)

(A) Remove release liner from pouch adhesive ring. (B) Fold pouch adhesive ring in half without touching adhesive ring. Place pouch adhesive ring onto the adhesive ring.

8. Insertion of the protector (Figure 7)

With adhesive ring facing outwards, place the pouch onto the skin barrier. Retain the etiquette adhesive during reapplication.

9. Removal of the pouch (Figure 8)

With adhesive ring facing outwards, gently pull the pouch away from the skin barrier.

10. Cleaning the pouch (Figure 9)

With adhesive ring facing outwards, gently pull the pouch away from the skin barrier.

11. Disposal

Please follow all local disposal regulations.

This device is for single-use only and should not be reused. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard, handle and dispose in accordance with accepted medical practice and applicable local laws and regulations.

If the use of this device or a result of its use has caused an incident, please report it to the manufacturer and to your national authority.

© 2024 ConvaTec Inc.
*™ indicates trademarks of Convatec Inc. ESTEEM SYNERGY and INVISCLOSE are registered trademarks of Convatec Inc. in the United States.

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5. The Empty Pouch Can Be Repositioned After Application (Figure 6)

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6. Closing The Pouch With Adhesive Collar (Figure 7)

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7. Emptying The Pouch

With adhesive ring facing outwards, gently pull the pouch away from the skin barrier.

8. Information About Pouches With Filters

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