## en ENGLISH

**INDICATIONS:** For use on skin around stomas and skin around wounds.

**DESCRIPTION OF DEVICE & CONTENTS:** ESENTA™ skin barrier spray sting free (50mL)

**INTENDED USE:** Protection of skin from the damage associated with body waste, enzymes and adhesives.

**TARGET PATIENT POPULATION:** People with stoma and wound dressings.

**INTENDED USER:** Health Care Professionals, Caregivers and Ostomates.

### INSTRUCTION FOR USE:

1. Clean and dry the skin.
2. Hold approximately 10cm from the area to be treated.
3. Apply a uniform coating in a sweeping motion.
4. Ensure the area is completely dry before applying adhesive products such as appliances or dressings.
5. Replace at every dressing or appliance change to maintain complete protection.

**INFORMATION:** Removal is not normally necessary. However, ESENTA™ skin barrier can be removed easily using ESENTA™ medical adhesive remover. Sprays at any angle. Soothes reddened and sore skin. Reduces itching. If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

### CAUTION:

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vaginal).
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- Avoid inhalation.
- For external use only.

**WARNINGS:** Use only as directed. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C / 122°F. Keep out of reach of children. EXTREMELY FLAMMABLE

Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations

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