en **ENGLISH**

Indications: For use on skin around stomas and skin around wounds.

DESCRIPTION OF DEVICE & CONTENTS: ESENTA™ skin barrier wipes sting free

INTENDED USE: Protection of skin from the damage associated with body waste, enzymes and adhesives.

TARGET PATIENT POPULATION: People with stoma and wound dressings.

INTENDED USER: Health Care Professionals, Caregivers and Osomates.

INSTRUCTION FOR USE:

- Clean and thoroughly dry the skin
 - Tear sachet to open and unfold wipe
 - Gently wipe the area to be protected and wait until dry
- INFORMATION:** Dries in a few seconds. Create a clean surface without affecting the adhesion of the next application or dressing. Soothes reddened and sore skin. Reduces itching.

CAUTION: This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations. If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- For external use only.

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